

INNOVATIVE H.A. FOR CARTILAGE MAINTENANCE



CONSERVATIVE INJECTABLE THERAPY FOR ARTICULAR CARTILAGE

WORLDWIDE NEW

INTRODUCTION

Osteoarthritis (OA) is a degenerative disease that affects different tissues of the surrounding joints such as articular cartilage, subchondral bone, synovial membrane, and ligaments. No therapy is currently available to completely prevent the progression of the disease due to poor understanding of the pathology mechanisms. Articular cartilage is the main tissue afflicted by OA, and chondrocytes are cellular components of the tissue that actively participate in the degeneration process. Multiple factors influence the development and progression of OA including inflammation during disease progression and by the biomechanical alterations of cartilage due to wear and trauma. During the progression of OA, extracellular matrix (ECM) cartilage is actively remodelled by chondrocytes in inflammatory conditions.

This alteration of ECM, in turn, changes the biomechanical environment of chondrocytes, which further push the progression of the disease in the presence of inflammation. Changes in the composition and ECM structure also prevent the participation of mesenchymal stem cells in the cartilage repair process by inhibiting their chondrogenic differentiation.

SINODRIAL

The Wave innovation group has been making injectable medical devices for the treatment of osteoarthritis diseases for years. Devices such as **SINODRIAL** not only slow down the degradation of articular cartilage, but thanks to a specific high molecular weight hyaluronic acid, with amino acids and precursors, they stimulate the production of elastic and reticular fibers that make up the cartilage.

SINODRIAL is an injectable class III Medical Device.

CONCLUSION

SINODRIAL demonstrates its clinical activity by activating the metabolic and reparative processes of articular cartilage. The high molecular weight of hyaluronic acid with precursors and amino acids, stimulate chondro regeneration with cell surface receptors, increase the production of type II collagen and minor collagens, and positively influence the ability of chondrocytes for the correct maintaining of the extracellular matrix and cartilage.



TECHNICAL PRODUCT SHEET

PRODUCT DESCRIPTION		Sterile injectable medical device for intra-articular use, is a synovial fluid replacer, helps restore the rheological conditions of altered joints. 2 pre-filled syringe, each containing 2ml/50mg of non-pyrogenic gel, sterilized using moist heat.		
TECHNICAL CHARACTERISTICS OF THE PRODUCT				
ACTIVE INGREDIENT		Sodium Hyaluronate 25 mg/ml with a molecular weight of 3 Million Daltons not crosslinked, with hidroxyproline, proline, glycine and lysine,...		
EXCIPIENTS		Sodium monobasic phosphate, sodium dibasic phosphate, sodium chloride, water for injection.		
PRIMARY PACKAGING MATERIAL		Crystal syringe in blister	VOLUME	2 ml
SECONDARY PACKAGING MATERIAL		Cardboard		
COLOR		Transparent	STERILITY	Sterile
LAW AND DIRECTIVES APPLIED		Medical Device Class III designed as Directive 92/43/CEE, manufactured in accordance with ISO 13485 requirements and regulation (EU) 2017/745.		
PRODUCT PURPOSE		SINODRIAL is a Bio-regenerant based on collagen precursors, which facilitates chondrocytes to produce quickly tropocollagen that polymerizes in the extracellular matrix of the cartilage forming real collagen. It is indicated for the conservative treatment of the cartilage and its annexes.		
HOW TO USE	PROTOCOL AND DOSAGE			APPLICATION APROACH
	At the discretion of the doctor	2/4 treatments per year (1 syringe every 2 weeks). Renew every year.		To inject in joint
FIELD OF APPLICATION		Recommended for use in traumatology (Osteoarthritis)		
CE MARKING		CE 0373		
PRODUCTION		3 months		
TRANSPORT CAUTIONS		Controlled temperature		
STORAGE CAUTIONS		Keep in a dry place, avoid moisture, keep away from sunlight and fluorescent light, store between +2° and +25° C.		
MINIMUM ORDER QUANTITY (MOQ)		2520 units		
EXPIRATION		36 months, after manufacture date		
TRADEMARK		SINODRIAL		
MANUFACTURER		THE WAVE INNOVATION GROUP SRL, ITALIA		

Contact person:
FRANSIN KELEN TERRANI
COMMERCIAL MANAGER
Cell. +39 392 542 2339
administration@thewaveinnovation.com

THE WAVE INNOVATION GROUP S.r.l.
Via Mons. Luigi Bellotti, 16
37139 Verona, Italy
www.thewaveinnovation.com
info@thewaveinnovation.com

